

Hyperoil®

INTERACTIVE WOUND DRESSING

for Acute - Chronic - Infected wounds



RI.MOS. | CARE

Hyperoil®

IN CASE OF

- WOUNDS
- ABRASIONS
- EXCORIATIONS
- BURNS
- RADIATION DERMATITIS
- VASCULAR LESIONS
- PRESSURE ULCERS

EXAMPLES



ABRASION



Day 0



Day 35

Acute phase:

OIL formulation twice a day.

Maturation phase:

GEL formulation twice a day.



RADIATION DERMATITIS



Day 0



Day 42

OIL formulation two hours before

and immediately after radiation therapy.



DECUBITUS ULCER



Day 4



Day 124

OIL formulation once a day.

Topical treatment that promotes wound healing
for external wounds

PLANTS WITH SYNERGISTIC PROPERTIES



NEEM (Azadirachtin)



Antiseptic
Antimicrobial
Anti-inflammatory



HYPERICUM (Hyperforin)

Tissue regeneration
Re-epithelialization
Lenitive



WHAT I GET WITH HYPEROIL[®]

PREVENTION



WOUND
MANAGEMENT



FAST ACTING



HOME CARE



QUALITY OF LIFE



Hyperoil is available in OIL, GEL and GAUZES formulation to treat a variety of skin injuries.

OIL FORMULATION - acute phase



50 ml Oil Glass dropping bottle
Ref.10050CGO



5 ml Oil Re-Sealable Vial
Ref. 100101HY



100 ml Glass Bottle
(with spray nozzle)
Ref. 100610



GEL FORMULATION - maturation phase



5 ml Gel Re-Sealable Vial
Ref. 100501HY



30 ml Gel Tube
(with applicator)
Ref. 10030TGO



100 ml Glass Bottle
(with spray nozzle)
Ref. 100100SPO



10 pcs (10x10 cm)
Medicated Gauzes
(individually packed)
Ref. 10010GMOS



The amount of product to be applied may vary depending on the wound that you are managing (size and depth).

It is recommended to apply the product over the edges of the wound, to ensure proper tissue regeneration.

Exceeding the recommended amount does not increase the effectiveness of the product.

Class IIb Medical Device

THIS BROCHURE IS EXCLUSIVELY INTENDED FOR PROFESSIONAL USE

Hyperoil® SCIENTIFIC LITERATURE

Buruli Ulcer

- Iabichella ML et al.(2014), *Austin J Clin Case Rep*, 1(7): 3

- Iabichella ML et al. (2015), *Angéiologie*, 67(1): 29-41

Diabetic Foot

- Iabichella ML (2013), *BMJ Case Reports*, doi: 10.1136/bcr-2012-007299

- Iabichella ML, Caruso C, Lugli M (2014), *BMJ Case Reports*

- Iabichella ML (2015), *J Diabetes Metab*, 6:2

Radiation Dermatitis

- Franco et al. (2014), *Radiation Oncology*, 9:297, doi: 10.1186/s13014-014-0297-0

- Franco P, Rampino M, Ostellino O, Schena M, Pecorari G, Garzino Demo P, Fasolis M, Arcadipane F, Martini S, Cavallin C, Airoidi M, Ricardi U. *Med Oncol* 2017; 34:30.

Severe Acute Malnutrition

- Iabichella ML et al. (2017), *Austin J Clin Case Rep*, 4(1): 1111

Systemic Sclerosis

- Giuggioli D. et al. (2020) *J Int Med Res*, 48(4):300060519882176.



RIMOS s.r.l.
Medical Products

Via Manuzio, 15
41037 - Mirandola (MO)

info@rimos.com - www.rimos.com

COD. 100810-E
REV. 04-2024